


How to anonymize a patient exam in Voluson

FASSprop-study

Instructions

- Press the “Archive button”
- Select “Local Archive”
- Select the patient in the list, the alternative “Anonymize” will appear below
- Press “Anonymize”
- Add the MediCase generated number as Patient-ID, Last name and First name
- Select “Anonymized Archive”
- Select patient
- Go to “Data transfer” on the right side of the screen and Select “Export”.
- Export as DICOM to a USB-stick

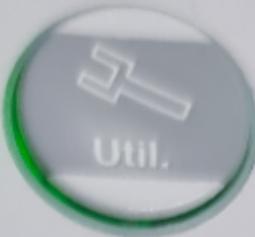

Source **Local Archive**

Search **Patient Name**

Patients & Exams

Exams Only

—

AND

OR

Exam Date

2021/03/26

Patients

B	Patient ID	Patient Name	Birthdate	E	Sex	↑ Last Exam
	VE10 BT17 0014	C1-5 Obstetrics		1		2016/09/19
	VE10 BT17 0012	C4-8 Obstetrics		1		2016/09/19
	VE10 BT17 0011	RAB6 Obstetrics		1		2016/09/19
	VE10 BT17 0010	RM6C Obstetrics		1		2016/09/19
	VE10 BT17 0009	Fetal Echo RAB6		1		2016/09/19
	VE10 BT17 0008	Fetal Cardio Demo		1		2016/09/19
	VE10 BT17 0007	HDlive Flow HDlive Flow ...		1		2016/09/19
	VE10 BT17 0006	HDlive Studio Demo		1		2016/09/19
	VE10 BT17 0004	eSTIC		1		2016/09/19
	VE10 BT17 0003	Gynecology Demo		1		2016/09/19
	VE10 BT17 0002	Radiance e4D Demo		1		2016/09/19

Local Archive

Search Patient Name

AND

Patients & Exams Exams Only

Exam Date

2021/03/26

Show

Patients

B	Patient ID	Patient Name	E	Sex	Last Exam
	VE10 BT17 0014	C1-5 Obstetrics	1		2016/09/19
	VE10 BT17 0012	C4-8 Obstetrics	1		2016/09/19
	VE10 BT17 0011	RAB6 Obstetrics	1		2016/09/19
	VE10 BT17 0010	RM6C Obstetrics	1		2016/09/19
	VE10 BT17 0009	Fetal Echo RAB6	1		2016/09/19
	VE10 BT17 0008	Fetal Cardio Demo	1		2016/09/19
	VE10 BT17 0007	HDIlive Flow HDIlive Flow ...	1		2016/09/19
	VE10 BT17 0006	HDIlive Studio Demo	1		2016/09/19
	VE10 BT17 0004	eSTIC	1		2016/09/19
	VE10 BT17 0003	Gynecology Demo	1		2016/09/19
	VE10 BT17 0002	Radiance e4D Demo	1		2016/09/19

Capacity: 400.76 GB
 Used: 142.12 GB
 Free: 258.64 GB

Nr. of Patients: 204 / 204



Selected Patients: 1
 Selected Exams: 1
 Selected Images: 19
 165.67 MB

Exams

Exam Lock:

B	Exam Date	Exam Time	Exam Type	M	Img	Application	Exam Size
	2016/09/19	14:58:20		Y	19	Obstetrics	165.67 MB



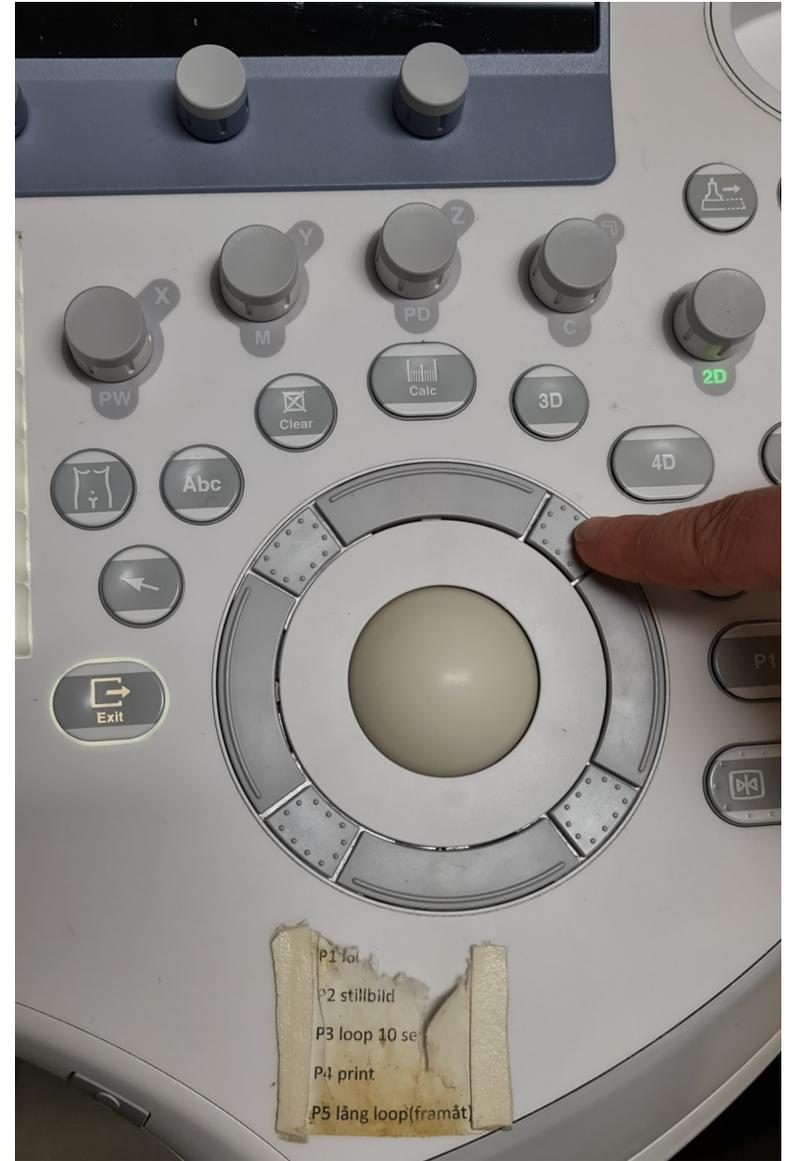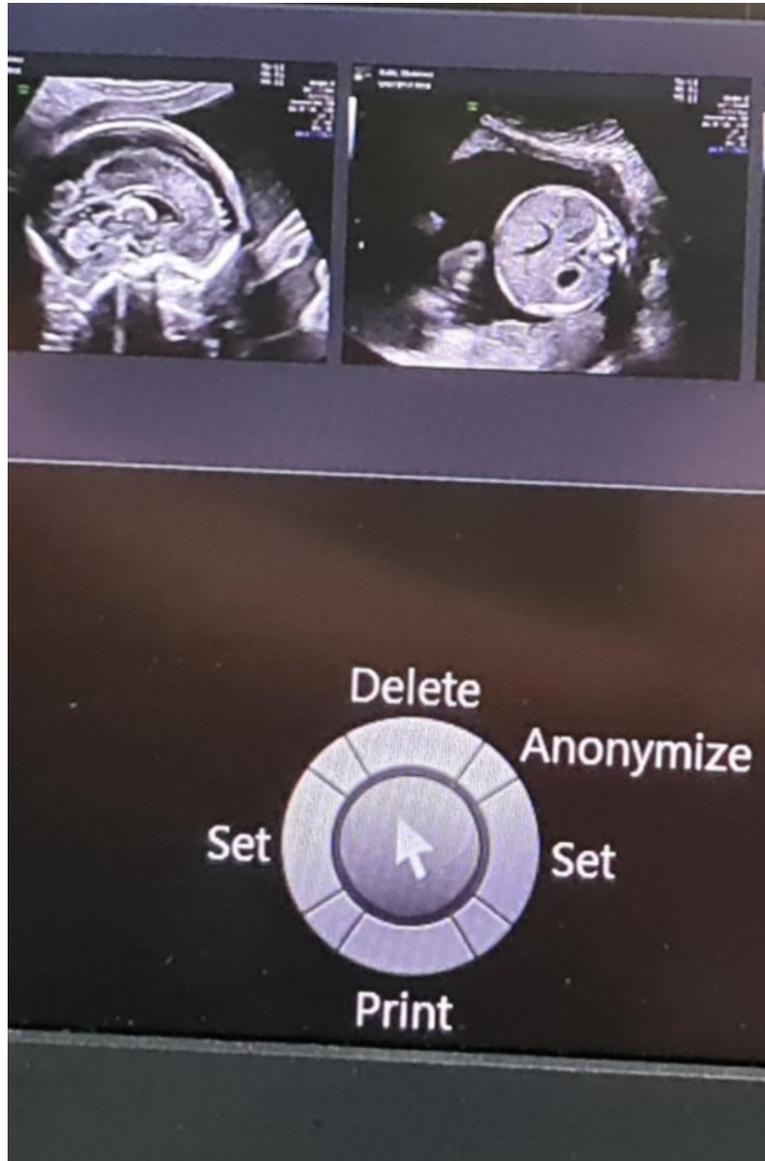



Anonymize

Original Patient Data:

Patient ID: VE10 BT17 0011

Name: RAB6

Last Name: Obstetrics

Middle Name:

Anonymized Patient Data:

Patient ID: [Redacted]

DOB: yyyy/mm/dd

Sex:

Age:

Last Name: [Redacted]

First Name: [Redacted]

Middle Name: [Redacted]

Exam Comment: [Redacted]

Exam Type: [Redacted]

Generate Patient ID automatically

Clear all

Cancel

OK

- Add the Medicase number as
- Patient-ID
- Last name
- First name



Source **Local Archive**

Local Archive
4DV

Patient Anonymized Archive only

Search **Patient Name**

AND

Exam Date

Patients

B	Patient ID	Patient Name	Birthdate	E	Sex	↑
	VE10 BT17 0014	C1-5 Obstetrics		1		
	VE10 BT17 0012	C4-8 Obstetrics		1		
	VE10 BT17 0011	RAB6 Obstetrics		1		
	VE10 BT17 0010	RM6C Obstetrics		1		
	VE10 BT17 0009	Fetal Echo RAB6		1		
	VE10 BT17 0008	Fetal Cardio Demo		1		
	VE10 BT17 0007	HDlive Flow HDlive Flow ...		1		
	VE10 BT17 0006	HDlive Studio Demo		1		
	VE10 BT17 0004	eSTIC		1		
	VE10 BT17 0003	Gynecology Demo		1		
	VE10 BT17 0002	Radiance e4D Demo		1		

Source **Anonymized Archive**

Search **Time**

Patients & Exams Exams Only

AND OR

Date **2021/03/26**

Patients

B	Patient ID	Patient Name	Birthdate	E	Sex	↑ Last Exam
	23	23 23		1		2021/03/26
	test25	Test25 Test25		1		2021/03/26
	ANONYM-00001	Z W		1		2020/12/01
	8939-2	8939-2 8939-2		1		2020/04/28
	121212	121212 121212		1		2020/03/27

Exams

B	Exam Date	Exam Time	↑ Exam Type	M	Img	Application
	2021/03/26	10:13:49 AM		Y	19	Obstetrics

Data Transfer

Export

Import

Print

DICOM Send


Send Email

Exit



1s



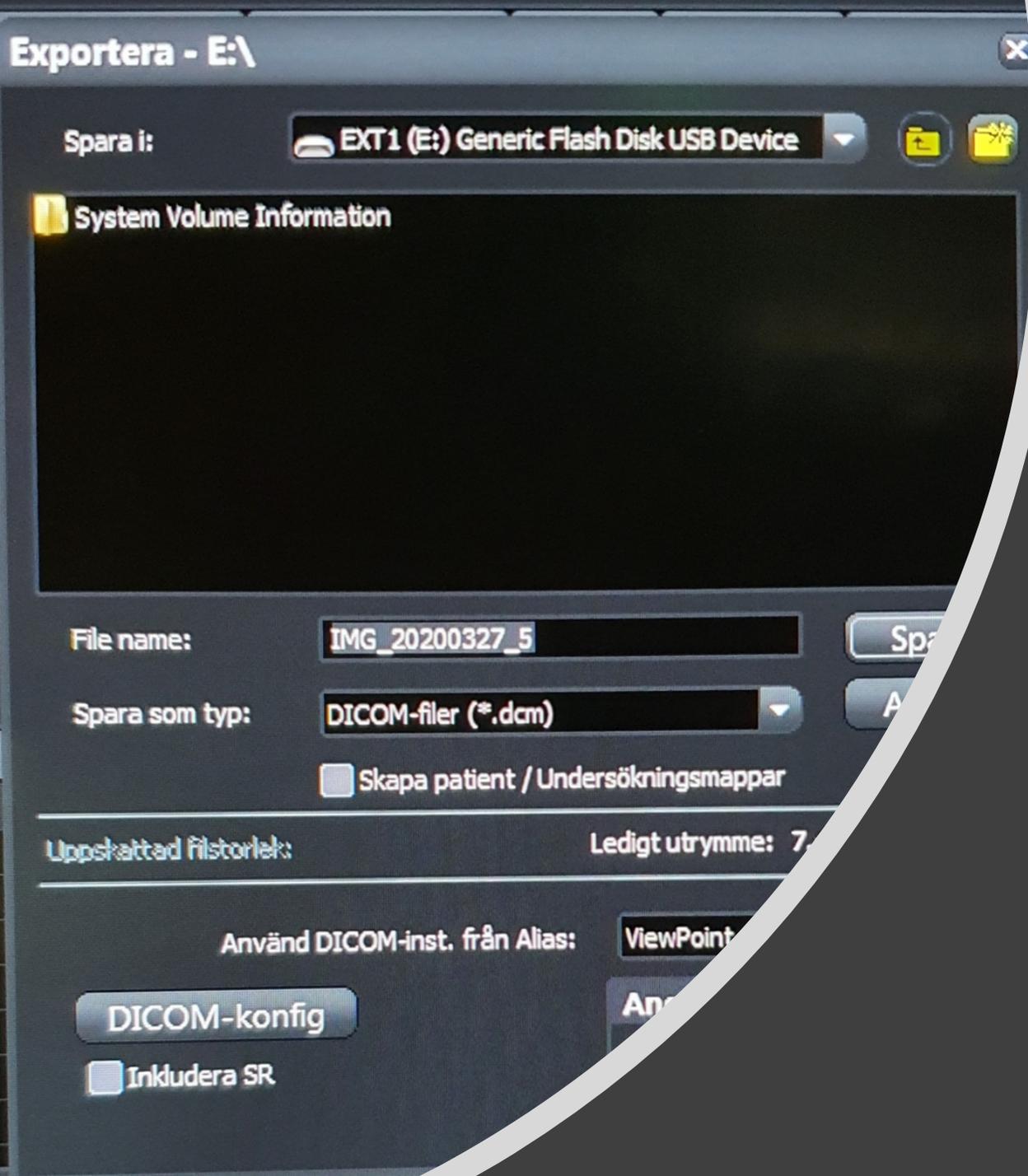

Export as DICOM
to USB (on the
right side of the
screen)



Save.
Eject the USB and
import the exam
to Filezilla